

# Option:

## "BZ7E/BZ8E: Electric leg rest"


For some wheelchairs there is the possibility to have electric leg rests.

For the instructions of the wheelchair refer to the instruction manual of the wheelchair.

### 1. Technical Specifications

Description	Dimensions
Weight	8,56 kg/pair
Length adjustment	185 mm
Depth adjustment	25 mm/side
Angle adjustment	10° - 95°

### 2. Components

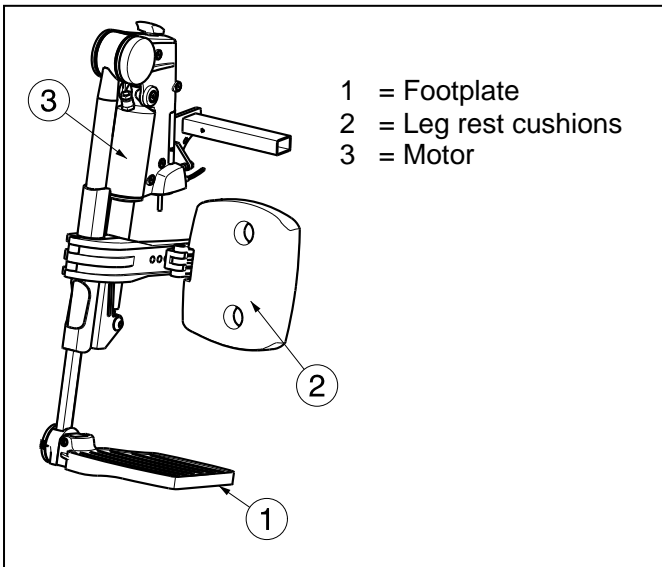

### 3. Safety rules

⚠ **Make sure that all screws are firmly secured before driving with your electric wheelchair.**

### 4. Manner of delivery

The electric leg rest shall be delivered with:

- Footplate
- Leg rest cushions
- Instruction manual


Before use check if everything is included and that no products are damaged (example by transport, ...).

### 5. Tools

Allen key set n° 4, 5  
Wrench set n° 10

### 6. Mounting / removing leg rest

⚠ **WARNING: Risk of injury - Make sure that the leg rests are firmly locked before using the leg rests.**

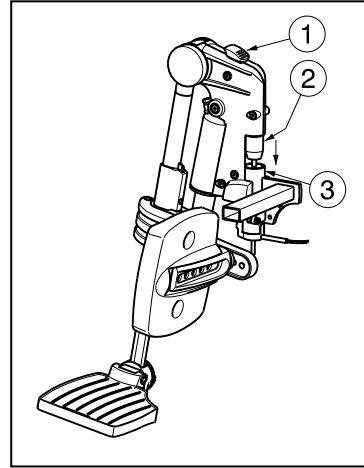

The mounting of the leg rests is done as follows:

1. Mount the jack male ② in the wheelchair attachment tube ③.
2. Check that the leg rest is firmly secured.

To take off the leg rests:

1. Push on lever ①.
2. Pull the leg rest upwards out the wheelchair attachment ③.

### 7. Angle of leg rest


⚠ **WARNING: Risk of injury - Make sure that no objects and/or persons are inside the swinging range of the adjust functions.**

The angle of the leg rest you can adjust with your operator control (range 85°: step less).

### 8. Length leg rest

⚠ **WARNING: Make sure that the leg rest is firmly secured.**

⚠ **CAUTION: Avoid that the leg rests make any contact with the ground. Keep a minimum distance from 60 mm above the ground.**

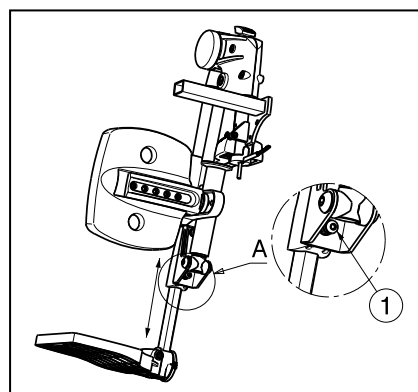


Adjust the length of the leg rests as follow:


1. Loosen the screw and nut ① a little bit (on the back of the leg rest).

2. Adjust the length of the leg rest to a comfortable length (range 185 mm: step less).
3. Tighten the screw and nut ① properly.

The max adjustable length of the leg rest shall be marked with a triangle on the tube.  
Not possible when leg rest is in vertical position (safe distance of 60 mm from the ground).

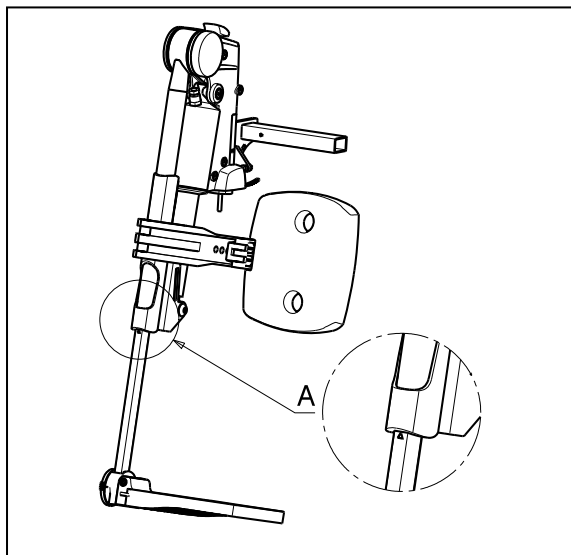

### 9. Adjusting angle foot plate

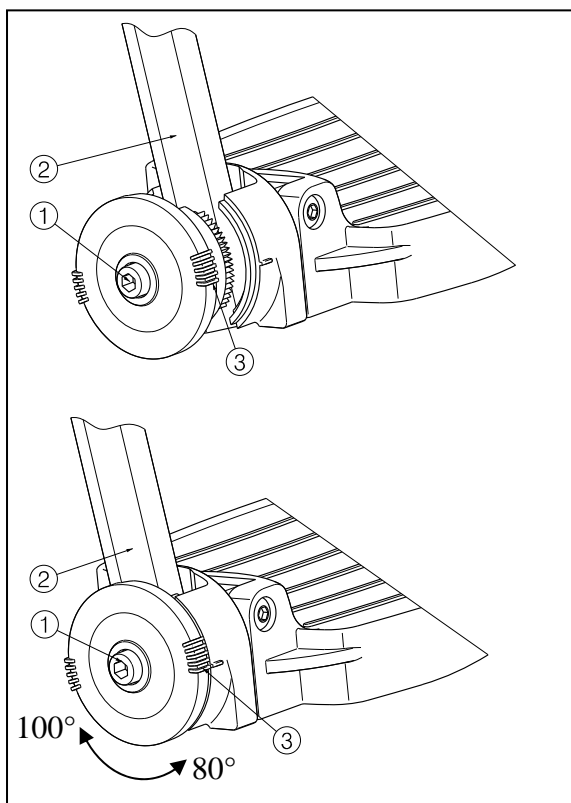


Adjust the angle of the foot plate as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (80°-85°-90°-95°-100°) is according the dashes ③.
3. Retighten the bolt ①.

### 10. Adjusting the width of the leg rest


**⚠ WARNING: Risk of injury - Make sure that all screws are secure and properly tightened before using the leg rests.**

**⚠ WARNING: Risk of injury - Make sure that the leg rests are positioned symmetrically from the seat.**

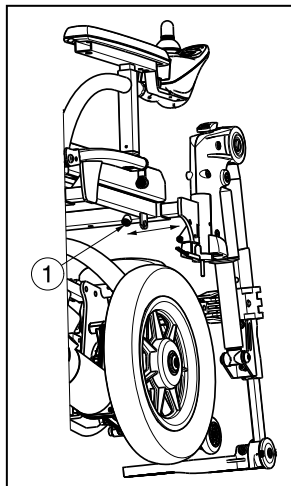


Adjust the width of the leg rests as follow:

1. Loosen the hexagon socket screw ① just under the seat frame.
2. Adjust the leg rest in the desired position.
3. Retighten the hexagon socket screw ①.
4. Repeat instructions above for the other leg rest.

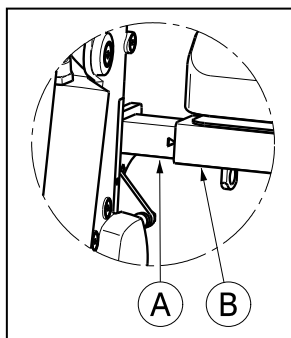


The maximum position of the seat width is marked with the triangle on the square tube ①. Place the point of the triangle against the square tube ② of the seat frame.

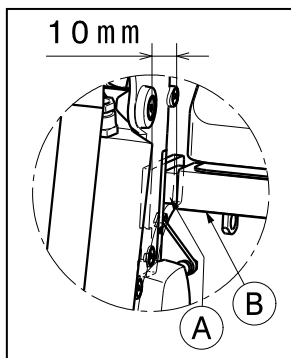


The minimum position of the seat width is with a safe distance of 10 mm between the square tubes ① and ②.

### 11. Fold away leg rest cushion

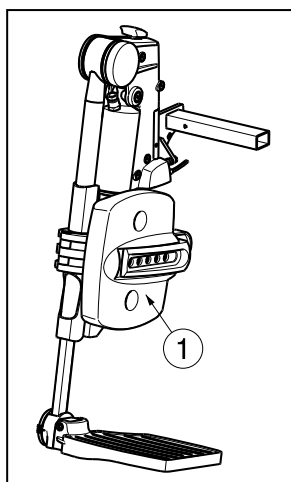

The leg rest cushion can be easily fold away as follow:

1. Take the cushion ① on the outside.
2. Fold the cushion ① to the front.

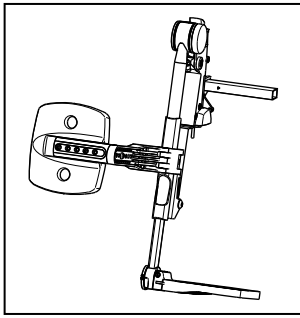


3. Take the cushion ① on the back.
4. Swing the cushion ① forwards.

To put the leg rest cushions in the begin position you can do the reverse.

**Do not place your fingers between the movable parts or where they can be squeezed.**


### 12. Adjusting height leg rest cushion

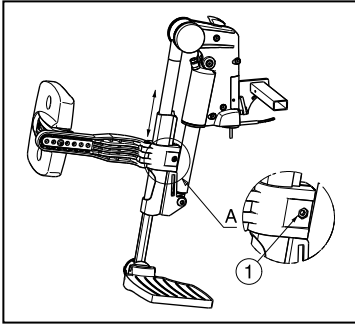

1. Fold away leg rest cushion (according previous paragraph).
2. Loosen bolt ① on the side of the leg rest.
3. Adjust the height of the leg rest to a comfortable position (range 60 mm: step less).
4. Retighten the bolt ①.


### 13. Adjusting depth leg rest cushion

1. Remove the screw ①.
2. Adjust the depth of the leg rest cushion to a comfortable position ( 7 different positions in steps of 15 mm).
3. Retighten the screw ①.

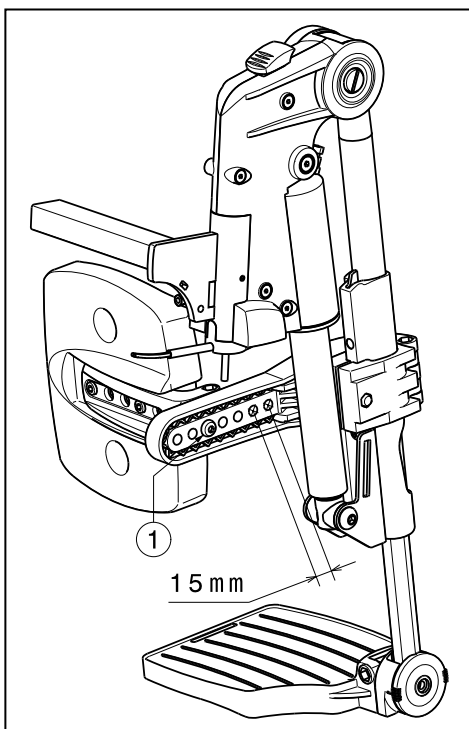

### 14. Adjusting width leg rest cushion

1. Remove the two bolts ① on the back of the leg rest cushion.
2. Adjust the width of the leg rest cushion to a comfortable position (2 different positions in steps of 20 mm, **do not use the middle hole ② for adjusting**).
3. Retighten the two bolts ①.

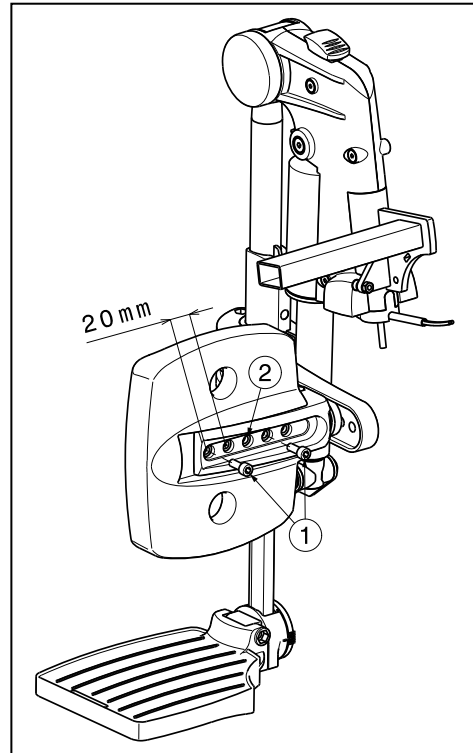

## Belgium

### N.V. Vermeiren N.V.

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

### Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

### Vermeiren Italia

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.vermeiren.it](http://www.vermeiren.it)  
e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## Poland

### Vermeiren Polska Sp. z o.o

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

### Vermeiren ČR S.R.O.

Nádražní 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 121 976  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

### Vermeiren Deutschland GmbH

Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

### Vermeiren Austria GmbH

Schärdinger Strasse 4  
A-4061 Pasching  
Tel: +43(0)7229 64900  
Fax: +43(0)7229 64900-90  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

### Vermeiren Suisse S.A.

Hühnerhubelstraße 59  
CH-3123 Belp  
Tel: +41(0)31 818 40 95  
Fax: +41(0)31 818 40 98  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

### Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)