


1903211

<b>LOWER FRAME ALESIA</b> <b>ONDERFRAME ALESIA</b>			Plan No. / Plannr.
			E000471/N
Scale / Schaal: 7:40	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd
Date / Datum	23/07/2008	26/07/2023	26/07/2023
Init. / Init.	SD	SD	PJ




Material / Materiaal

No. / Nr.	Plan	Article / Artikel	Number / Aantal
1	X000862	1006463	2
2	X001016	1001682	3
3	X001119	1001554	2
4	X004395	1902815	1
5	X004037	1902690	4
6	X004078	1699300	2
7	Y006626	1690574	1
8	Y006980	1690674	1
9	Y007015	1691101	1
10	Y008570	1691400	1
11	Y021868	2023501	1
12	---	1912500	1
13	Y021871	2023401	1
14	Y013418	1691074	1
15	---	1001557	2
16	---	1912501	1
17	---	1990648	4
18	---	1001610	2
19	---	1001616	2
22	---	1001500	2
23	---	1900008	1
24	---	1902149	2
25	---	1902150	2
26	---	1902024	2
27	M8X20 DIN912	1906425	2
28	M8X12 DIN912	1008963	2
29	---	1992349	2
30	---	1992835	4
31	---	1992973	2
32	---	1993362	4
33	---	1994473	1
34	X004393	1700100	1
35	---	1902846	2
36	X004267	1903027	1
37	---	1998550	2
38	---	1911604	2
39	---	1912499	2
40	X004295	1903132	1
41	---	1009994	1
43	---	1001683	1
44	M5 DIN125	1001550	1
45	M8 DIN127	1001570	4

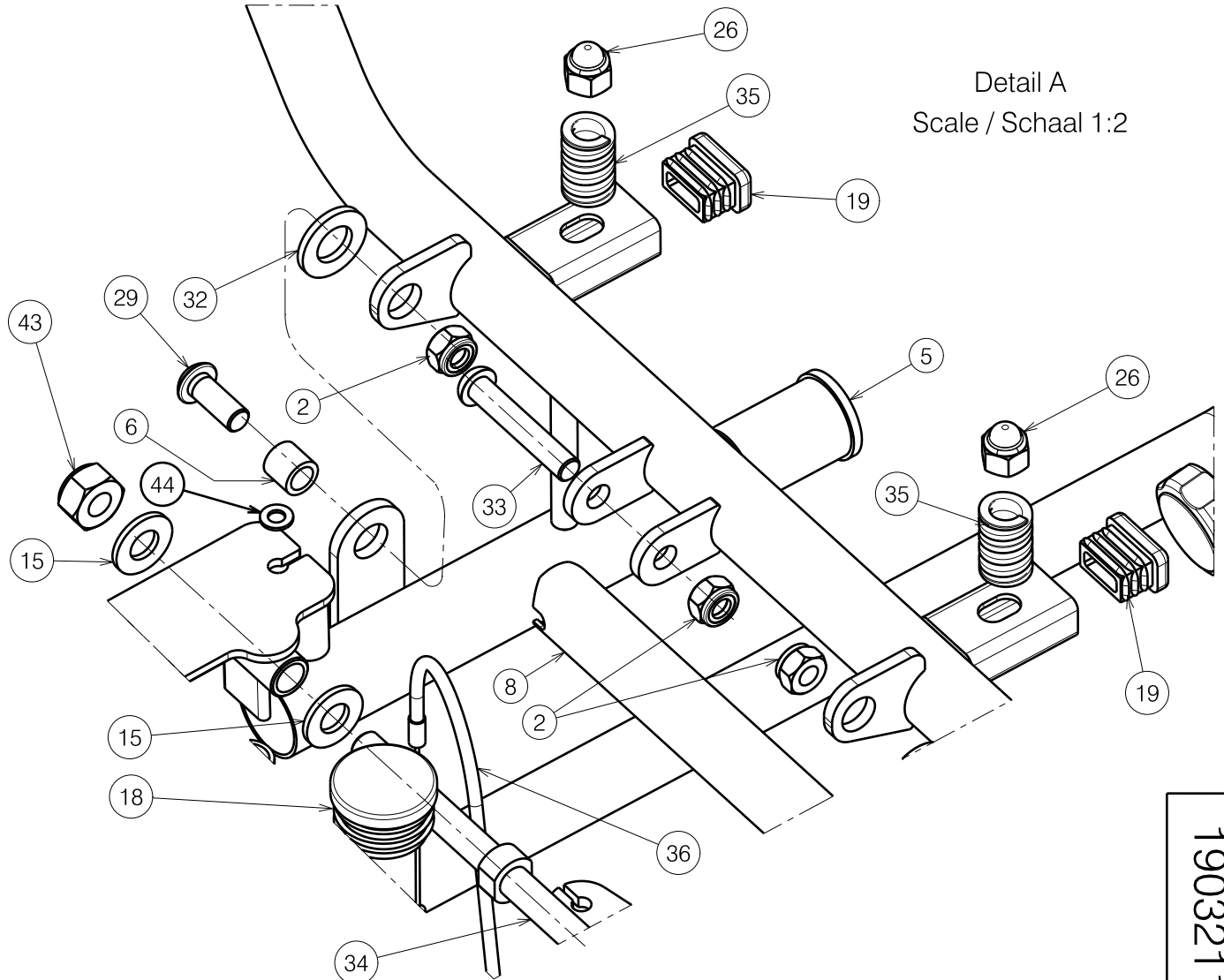

Detail A  
Scale / Schaal 1:2

1903211

LOWER FRAME ALESIA ONDERFRAME ALESIA				Plan No. / Plannr.
				E000471/N
Scale / Schaal: 7:40	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	23/07/2008	26/07/2023	26/07/2023	
Init. / Init.	SD	SD	PJ	