


Material			
No.	Plan	Article	Quantity
1	YP06157	2791882	1
2	YP06164	2794482	1
3	YP06167	2794682	1
4*	YP06196	2795082	4
5	YP06216	2784803	2
6	YP06217	2795303	2
7	XP00106	2990072	2
8	XP01954	2993706	4
9	Wheel D75	1001255	2
10	M6 DIN 985	1001680	4
11	M10 DIN 985	1001683	4
12	D8,4 DIN 125	1003199	4
13	3.9x16 DIN7982	1005767	4
14	M6x20	1006999	4
15	Velcro 12cm	1990663	0,5m
16	Wheel D75 with brake	1995807	2
17	M10x55	1998518	4
18	Velcro Strap 30cm	2993635	4

* - only for stacking beds / tylko do pietrowania lozek

Velcro straps (No.18) - way of assembly for YP06157 and YP06167 /
Paski Velcro (nr. 18) - sposob montazu dla YP06157 i YP06167

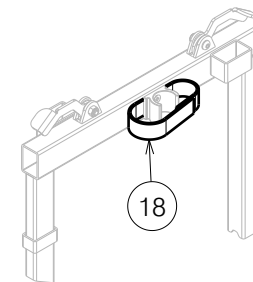


Package with bed dimensions [cm]		
Bed siderails	standard	965
Length	105	105
Width	44	47
Height	140	146
Carton box	U000369	U000367
Stacked beds height	283	289

EP00269 / H


TRANSPORT SYSTEM ILLICO				Plan No.
				1/2 EP00269/ H
Scale: 1:8	Drafted	Verified	Confirmed	
Date	09/01/2018	13/03/2024	13/03/2024	
Initials	DR	MK	AZ	

1

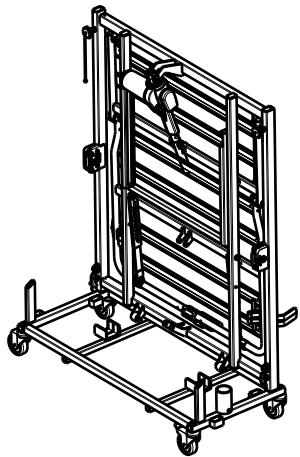


Mount the footside frame /
Zamontuj rame podnozka

2

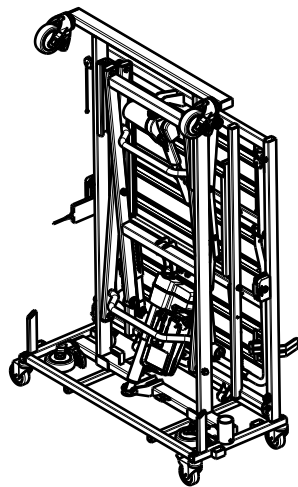


Mount the undercarriage /
Zamontuj rame podnosnika

3

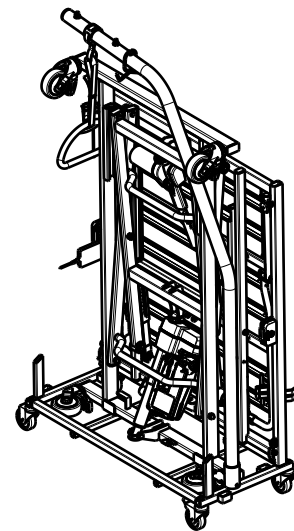


Mount the lifting pole / Zamontuj wysiegnik

4

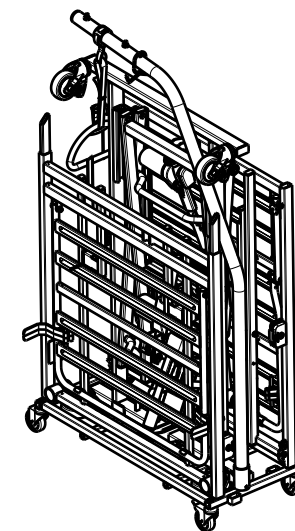


Mount the headside frame /
Zamontuj rame zaglowka

5

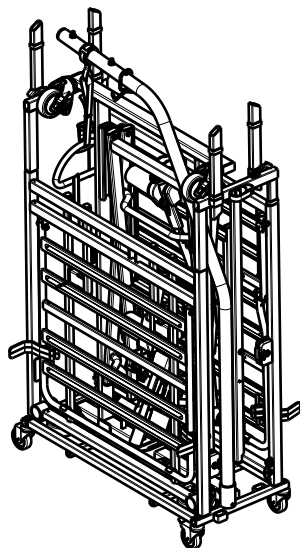


Mount the connectors /
Zamontuj laczniki systemu trans.

6

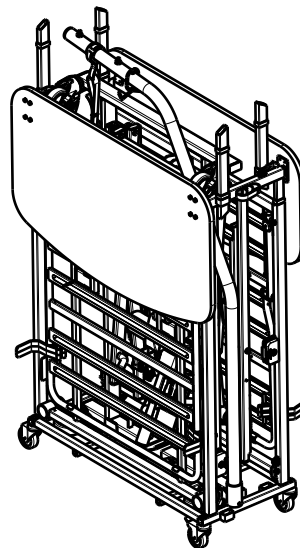


Mount the panels /
Zamontuj panele

7

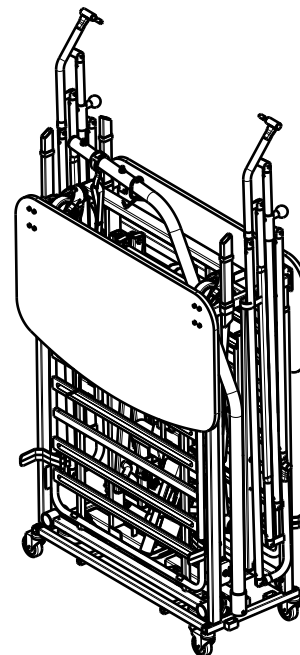


Mount the side rails
/ Zamontuj barierki

8

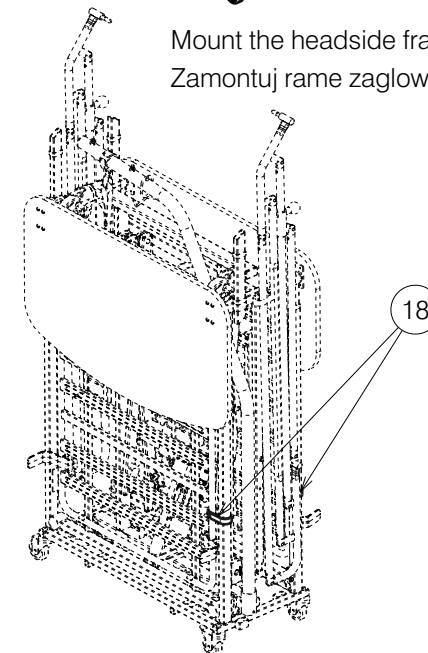

Put on the velcro straps to lock foot and headrest /
zaloz rzepy do blokady podnozka i zaglowka

EP00269 / H

TRANSPORT SYSTEM ILLICO				Plan No.
				2/2 EP00269 / H
Scale: 1:8	Drafted	Verified	Confirmed	
Date	09/01/2018	13/03/2024	13/03/2024	
Initials	DR	MK	AZ	